LABORATOIRES URGO

Fiche Technique N° : FT033

Edition N°: 8

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FICHE TECHNIQUE

PANSEMENT STERILE NON-TISSE BLANC EXTENSIBLE – *STRETCH WHITE NON WOVEN STERILE DRESSING*

URGOSTERILE

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- ↳ SERVICE QUALITE
- ↳ SERVICE A.R.E. URGO MEDICAL
- ↳ RESEAU PRESCRIPTION
- ↳ CHEF DE PRODUIT
- ↳ EXPORT

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□ DESCRIPTION:

- Extensible non-woven backing made up with white polyester fibres.
- Acrylic adhesive mass with high-skin tolerance.
- High absorption capacity non-woven central pad, covered by a polyethylene non-adherent net.
- Siliconed-paper protective wings.
- Adhesive on all 4 sides.
- Urgosterile is presented in a sterile (EO sterilised) peelable, individual and single-use sachet

□ PROPERTIES:

- Stretched: follows the movements of joints or skin wrinkles without rising rupture or lift-off.
- Non-woven: micro-aerated, it avoids maceration.
- High skin tolerance: respects sensitive skins,
- Does not adhere to the wound thanks to its polyethylene net covering the pad.
- Strong adhesiveness with easy removal, it protects the wound.
- Sterile dressing, it reduces risks of contamination.
- X-ray transparent.

□ INDICATIONS:

- Dressings for ambulatory surgery, coeliosurgery, post-surgery.
- Fast efficient and aseptic protection of puncture points: parenteral feeding, biopsy, dialysis.

□ INSTRUCTIONS FOR USE:

- Avoid tensions while applying.
- Apply on a clean dry skin (Using oily products such as creams can reduce adhesiveness).
- If an antiseptic is firstly used, rinse the wound carefully with a saline solution and dry the skin before applying URGOSTERILE.
- Check and change regularly the dressing.
- If you suspect an infection, consult a doctor.

□ PRECAUTIONS FOR USE:

- Do not expose to light for prolonged periods.
- Do not re-sterilise.

□ TECHNICAL DATA:

		SPECIFICATIONS
BACKING	Composition	Non-woven white polyester
ADHESIVE MASS	Chemical nature	Acrylic Emulsion mass with High skin tolerance
COATED BACKING	impregnated weight	35 to 45 g/m ²
	Total weight	77 à 97 g/m ²
	Adhesiveness	1.8 to 3.5 N/cm
	Water vapour permeability	Mini 1000 g/m ² /24h
PAD	Chemical nature	Viscose, covered by a non-adherent polyethylene veil.
	Impregnation	None.
	Average Absorbency*	≥ 580 %
PROTECTIVE PAPER	Chemical nature	One-side silicone paper
SIZES	Length	Mini 98 % of nominal value.
	Width	if nominal value ≤ 50 mm : ± 1,5 mm if nominal value > 50 mm : ± 2,5 mm
STERILITY	Sterilisation verification	Conformity of the sterilisation parameters and biological indicators
SHELF LIFE		5 years

* for information

□ REGULATORY STATUS :

CE marked Medical Device

*URGOSTERILE is distributed by:
Laboratoires URGO - 21300 CHENOVE - France*